

NAIL FUNGUS PEN

FUNGEX® NAIL FUNGUS PEN

- Proven effective formula against nail fungus
- Kills fungus on the nail within 20 seconds!
- 100% effective against nail fungus
- Prevents spread of the infection and helps to prevent re-infection

General advice: Before using FungeX® Nail Fungus Pen, read the instructions carefully, even if you have used this medical device before. The information contained in a new pack may have been revised in order to reflect the latest scientific insights and data. Nevertheless, you should use FungeX® Nail Fungus Pen with care in order to obtain good results. Keep these instructions – you may need to refer to them during treatment.

Application: FungeX® Nail Fungus Pen is a medical device that helps in the treatment of fungal nails (onychomycosis). FungeX® Nail Fungus Pen has been proven to be effective against fungal nail infections. The unique formula containing an 'INSTANT KILLER', contains ingredients that penetrate the surface of the nail, killing fungi within 20 seconds. The liquid forms a layer on the nail, creating an environment that is unfavourable to fungi growth and acting as a physical barrier to prevent the spread of moulds.

To achieve faster, better results, carefully filing the fungal nail once a week is recommended. If treatment is stopped, the fungal nail will return more quickly. It is therefore important to continue using FungeX® Nail Fungus Pen until the fungal nail has gone.

As damp environments are one of the causes of fungal nail infections, it is strongly recommended that you wear socks made only of absorbent materials, such as cotton or wool. This is important, as moulds are commonly found in moist environments, where they can multiply. If you have a fungal nail, it is important that you keep your feet dry.

Dosage and use (for adults only): FungeX® Nail Fungus Pen is for external use only. Apply the product to the affected area twice (mornings and evenings, for instance) until the fungal nail has fully grown out. Thanks to the 20-second instant kill formulation, the applicator Pen does not become infected with the fungi, preventing infection/reinfection of the other nails and protecting against any return of the moulds. FungeX® Nail Fungus Pen can be combined with the use of oral antimycotics.

Mode of action: Thanks to a unique combination of ingredients, FungeX® Nail Fungus Pen forms a layer, which creates an unfavourable environment for the growth of fungi in the nail plate. It also forms a physical barrier that inhibits the spread of moulds. FungeX® Nail Fungus Pen reduces the nail's acidity, creating an unfavourable environment that stops the growth and spread of fungi.

Use during pregnancy and/or breastfeeding: No information is available regarding the use of FungeX® Nail Fungus Pen during pregnancy or the breastfeeding phase. If you are pregnant or breastfeeding, consult your doctor or pharmacist before using.

Possible side effects: Possible side effects includes: erythema, irritation, edema, pain, nail discoloration and onycholysis; Allergic reactions.

⚠ WARNINGS

- For external use only.
- Fungal nail might cause erythema, irritation, edema, pain, nail discoloration and onycholysis.
- Check the list of ingredients to know if you are allergic to any of the ingredients contained in the gel.
- Avoid contact with eyes or mucous membranes.
- Do not apply to open wounds or damaged skin.
- Do not bandage tightly.
- Do not apply heat to the area of application.
- Pregnant or breastfeeding patients should seek advice of health professional before using Fungal nail.
- If you experience allergic reactions, stop using the product.
- Keep out of reach of young children.
- In case of no effect, consult a physician.

Contains: 1 FungeX® Nail Fungus Pen with 4 ml

Ingredients: Aqua, Ethoxydiglycol, ISK 1320®, Pentylene Glycol, Alcohol Denat., Tridiceth-9, PEG-40 Hydrogenated Castor Oil, Benzyl Alcohol, Limonene, Glycerin, NatiLact, Propylene Glycol, PVP, Xanthan Gum, Eucalyptus Globulus Leaf Oil, Lavandula Hybrida Oil, Linalool, Dimethyl Isosorbide, Citrus Aurantium Dulcis Oil, Pogostemon Cablin Oil, Disodium Phosphate, Geraniol, Citral, Biotin, Coumarin, Citric Acid, Farnesol.

All enquiries about the product should be sent to the distributor. For more information about FungeX® Nail Fungus Pen, please visit www.fungex.com

FungeX® is a registered trademark of TheOTCLab. FungeX® Nail Fungus Pen is a class 2A medical device according to EU regulation 93/42/EC.

 Do not use after expiry date (see packaging)

Read the instructions for use before using this medical device.

DISTRIBUTOR

TheOTCLab
Fred. Roeskestraat 115
1076 EE Amsterdam
The Netherlands

MANUFACTURER

PK Benelux BV
Vluchtoord 17
5406XP Uden
The Netherlands

Latest revision of text : December 2021



CE 0481

AF2100TI / V1.0